



SARA CE Front Panel

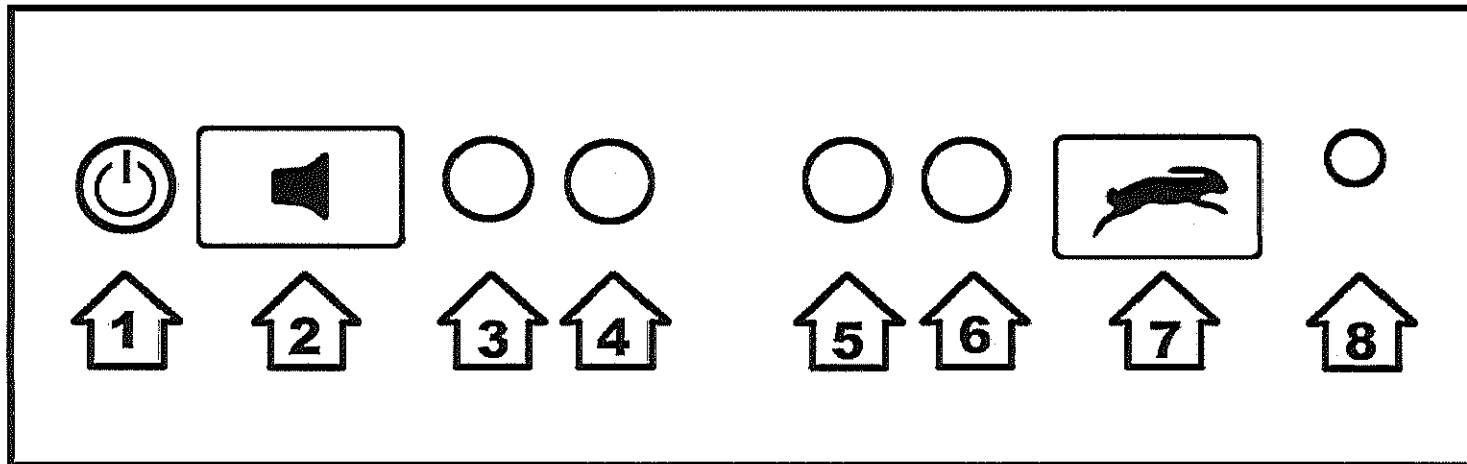

1. **Power button** – Turn SARA CE on or off.
2. **Volume Rocker** – Raise or lower the volume of the speech.
3. **Function Key 1** – Toggle the camera light on or off.
4. **Function Key 2** – Erase the current page.
5. **Function Key 3** – Cycle through scanning modes.
6. **Function Key 4** – Toggle column recognition on or off.
7. **Voice Rate Rocker** - Increase or decrease the reading speed.
8. **Headset Jack** – Connect the included headset.

SARA CE Top Panel

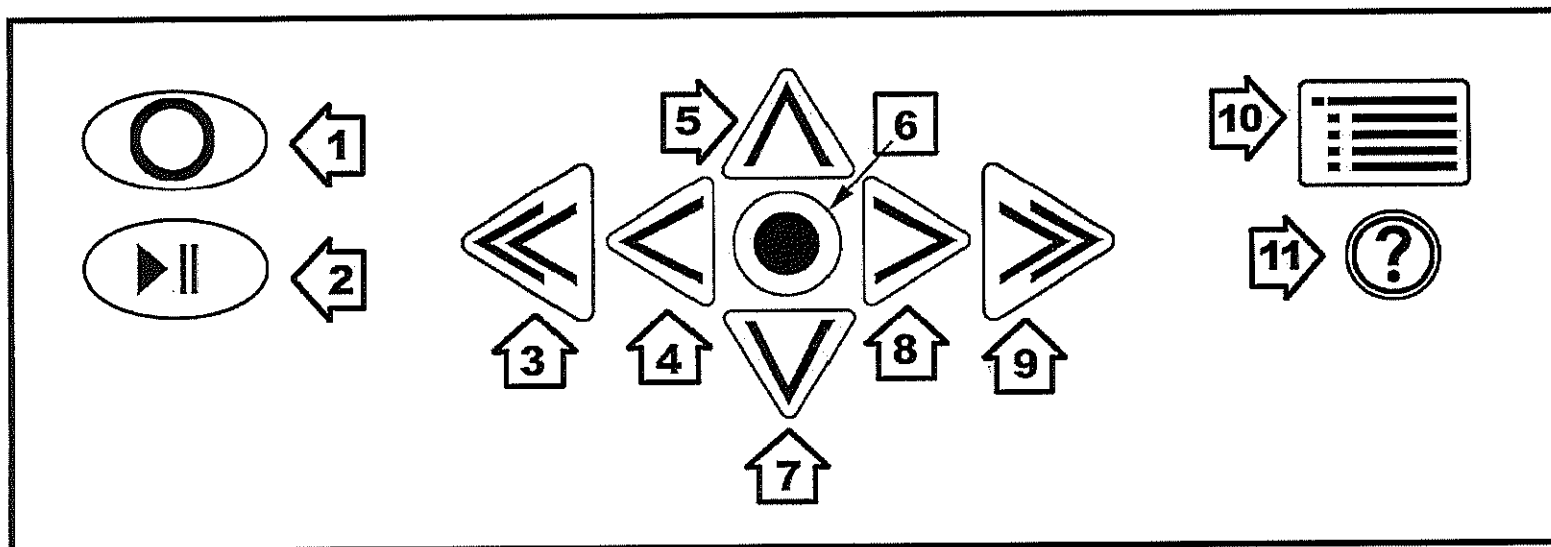

1. **Scan/Record Key** – Scan a document or record a file name.
2. **Play/Pause Key** – Start or stop reading.
3. **Left Double Arrow** – Move back one paragraph.
4. **Left Arrow** – Move left one word.
5. **Top Arrow** – Move up one line.
6. **Select Key** – Read current word.
7. **Down Arrow** – Move down one line.
8. **Right Arrow** – Move right one word.
9. **Right Double Arrow** – Move forward one paragraph.
10. **Menu Button** – Opens the menu.
11. **Help Key** – Press this key followed by any key to learn its function.